



NAME OF TRAINEE:

HELEN MOIG

POSITION:

SPECIALIST NURSE TRAINER
(CWD)

DATE:

13/05/15

TRAINED / ASSESSED BY:

Yasmine Abdullah

POSITION:

Exec. TM.

PUMP SOFTWARE VERSION:

2.1

KNOWLEDGE LEVEL

1 - Level required for a patient or those requiring a basic understanding of functionality and application of the product.

2 - Level required for a healthcare professional or key individuals and link nurses to ensure they are competent in the safe use and features of the product, troubleshooting and gained increased knowledge/skills.

3 - Level required by those responsible for training others. Those who are expected to facilitate teaching sessions with their enhanced product knowledge.

TRAINEE - SELF ASSESSMENT

I agree with the above assessment and I accept responsibility associated with the safe use of this device, to knowledge level 3.

Trainee Signature:

Helen Moig

Print Name:

HELEN MOIG

Date:

13/05/15

TRAINER/ASSESSOR VERIFICATION OF ATTENDANCE

The participant has been trained and assessed and the participant has reached the attained competency level as indicated.

Trainer Signature:

Y. Abdullah

Print Name:

Y. Abdullah

Date:

13 MAY 2015

TRAINING OUTCOME: PASS

Performance Criteria	Knowledge Level Required (1-3)	Evaluation Method	Competent (Yes / No)	Reference Instructions for use booklet item code: 7345771/EN01042	Notes / Comments
1 List the usual applications and main features of the Amika® enteral nutrition pump	1, 2 & 3	Direct questioning	✓	Pages 9 - 13 instruction for use booklet	
2 Identify the following features and components of the device: a. LED display b. Multi-function key pad c. Pump holder d. Pump door and release e. Battery f. Attachment to stand g. Cable fixing	1, 2 & 3	Observation / direct questioning	✓	Pages 9 - 21	
3 Keypad functions - a. On/off b. start/stop c. Progress bar (indicating volume delivered) d. Automatic priming e. Flow rate f. Target volume g. Alarm volume and mute h. Keypad lock i. Clear the progress bar	1, 2 & 3	Observation / direct questioning	✓	Pages 22 - 26 & 34	
4 Giving Sets a. Identify features of the giving set & functions b. Demonstrate inserting the giving set c. Demonstrate priming the giving set using the automatic prime function	1, 2 & 3	Direct observation & questioning	✓	Pages 23 - 25, 27 & 33	
5 Demonstrate the ability to set the rate of feed to be delivered and set the target volume to be infused	1, 2 & 3	Direct observation & questioning	✓	Page 29	
6 Ability to start and stop feeding and turn off pump	1, 2 & 3	Direct observation & questioning	✓	Pages 30 - 32	
7 Demonstrate the ability to check the volume that has been delivered	1, 2 & 3	Observation	✓	Pages 14 & 30	
8 Demonstrate ability to identify alarm conditions, audible and visual alarm and troubleshooting	1, 2 & 3	Direct questioning	✓	Pages 50 - 56	
9 Demonstrate ability to identify and understand the use and operation of back pack	1, 2 & 3	Observation	N/A	Instructions for use within backpack	
10 24 Hour pump helpline	1, 2 & 3	Direct questioning	✓	Refer to Amika® pump quick reference and pictorial guide*	
11 Cleaning and maintaining the pump	1, 2 & 3	Direct questioning	✓	Pages 45 - 47	
12 Demonstrate the ability to understand and use the menu functions	2 & 3	Observation	✓	Page 35 - 46	